

912023/03

Cellusan Light

Lubricating Eye Drops - Preservative free

Carboxymethylcellulose sodium (Carmellose Sodium, CMC) 0.5% w/v

30 reclosable vials of 0.4 ml

Composition: Carboxymethylcellulose sodium, lactic acid, sodium hydroxide, sodium chloride, potassium chloride, calcium chloride dihydrate, water for injections.

What is it?

Cellusan Light is a sterile, preservative free 0.5% w/v eye drops solution of Carboxymethylcellulose sodium (Carmellose Sodium, CMC).

How is presented?

Cellusan Light is presented as an eye drops, solution in 30 reclosable vials of 0.4 ml.

When should it be used?

Cellusan Light is used to obtain long lasting and soothing relief from mild to moderate dry eyes. It also provides protection against burning, irritation and discomfort associated with ocular dryness or exposure to environmental factors. The product is preservative free and suitable for all contact lenses. It may also be placed directly on the contact lenses to make the insertion much more comfortable.

How much to use?

1 drop in the affected eye(s) 2-3 times daily as required.
1-2 drop directly on the contact lenses before application.

Precautions and warnings:

- The product is for external ophthalmic use only.
- Do not use if the container is damaged.
- Do not use if you are sensitive to any of the ingredients.
- Do not use after the expiry date.
- Do not touch the eye with the tip of the vial.
- After administration, in rare cases, a slight temporary blurring of vision may be observed due to the viscosity of the solution; it is therefore advisable to wait for a few seconds before any activity that requires clear vision.
- Wait at least 10-15 minutes before instilling any other ophthalmic drug.
- Once opened each vial can be used for up to 12 hours.
- Keep out of the sight and reach of children.
- Do not swallow.
- Store below 25°C.

Manufacturer: Farmigea S.p.A. - Via G.B. Oliva 6/8 - 56121 Pisa, Italy

Date of last revision of the text: December 2013

CE 0546

STERILE A